



**HYPERTONIC SEA WATER NASAL SPRAY
CLEANSES & UNBLOCKS**



Sinomarin® Mini Spray 30 mL

STERILIZED PRODUCT

Children and adults (3 years and older)

DESCRIPTION: Nasal spray pump for children and adults, with anatomically designed nasal applicator. Metered-dose spray.

DOSAGE: 1 to 3 sprays per nostril, 3 times daily or as needed. Wash the nasal applicator after each use.

MODE OF ACTION: Sinomarin® contains naturally sourced hypertonic sea water (2.3% sodium chloride) which is rich in trace elements and mineral salts. It ensures an optimal cleaning of the nose and helps maintain healthy sinuses by removing excess mucus.

DIRECTIONS FOR USE: Remove the protective cap and follow the technique described below.

TO SELF-ADMINISTER OR TO ADMINISTER SINOMARIN® TO A CHILD

Blow your nose, or the child's nose, to clear out mucus before using Sinomarin®. Remove the protective cap, bend the neck gently forward and turn the head to one side. Insert the applicator into the upper nostril not more than ~ 0.6 cm (¼ inch) and press briefly. It is recommended not to breathe in when spraying; breathe through the open mouth so that the solution will not drain from behind the nose into the mouth. Do not swallow. Blow your nose, or the child's nose. Repeat the same process for the other nostril after turning the head to the opposite side. Wash the nasal applicator after each use.

Sinomarin® Children 100 mL

STERILIZED PRODUCT

Children (3 - 13 years old)

DESCRIPTION: Gentle spray for children, with anatomically designed nasal applicator.

DOSAGE:
Children 3-5 years old: One (1) spray per nostril, 3 times daily or as needed.
Children 6-13 years old: Two (2) sprays per nostril, 3 times daily or as needed.
Wash the nasal applicator after each use.

MODE OF ACTION: Sinomarin® contains naturally sourced hypertonic sea water (2.3% sodium chloride) which is rich in trace elements and mineral salts. It ensures an optimal cleaning of the nose and helps maintain healthy sinuses by removing excess mucus.

DIRECTIONS FOR USE: Remove the protective cap and follow the technique described below.

TO ADMINISTER SINOMARIN® TO CHILDREN

Blow child's nose to clear out mucus before using Sinomarin®. Remove the protective cap, bend the child's neck gently forward and turn their head to one side. Insert the applicator gently in their upper nostril not more than ~ 0.6 cm (¼ inch) and press briefly. Instruct the child not to breathe in when spraying or to breathe through their open mouth so that the solution will not drain from behind the nose into their mouth. Instruct the child not to swallow. Blow their nose. Repeat the same process for the other nostril after gently turning the child's head to the opposite side. Wash the nasal applicator after each use.

Sinomarin® Adults 125 mL

STERILIZED PRODUCT

Adults and adolescents (14 years and older)

DESCRIPTION: Medium spray for adults, with anatomically designed nasal applicator.

DOSAGE: Two (2) sprays per nostril, 3 times daily or as needed. Wash the nasal applicator after each use.


MODE OF ACTION:

Sinomarin® contains naturally sourced hypertonic sea water (2.3% sodium chloride) which is rich in trace elements and mineral salts. It ensures an optimal cleaning of the nose and helps maintain healthy sinuses by removing excess mucus.

DIRECTIONS FOR USE: Remove the protective cap and follow the technique described below.

TO SELF-ADMINISTER SINOMARIN®

Blow your nose to clear out mucus before using Sinomarin®. Remove the protective cap, bend your neck gently forward and turn your head to one side. Insert the applicator into the upper nostril, not more than ~ 0.6 cm (¼ inch) and press briefly. Do not breathe in when spraying; breathe through your open mouth so that the solution will not drain from behind your nose into your mouth. Do not swallow. Blow your nose. Repeat the same process for the other nostril after turning your head to the opposite side. Wash the nasal applicator after each use.

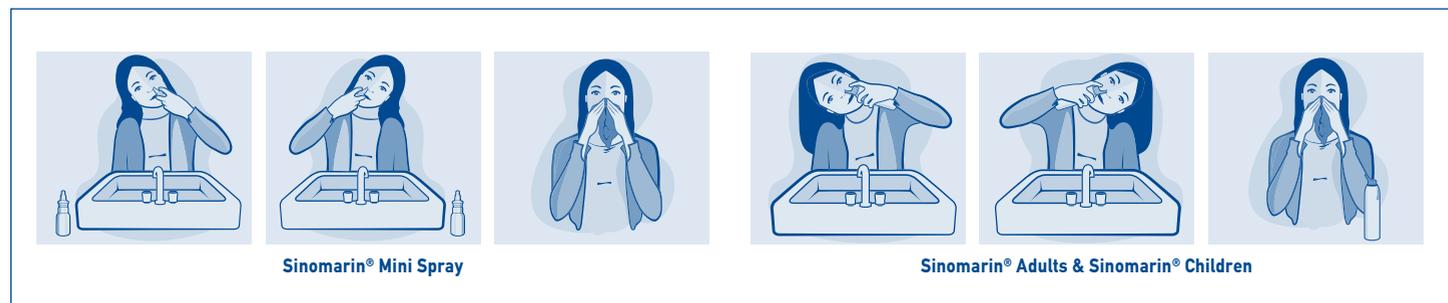

Preservative-free. Environmentally-friendly.

USE:

Clears blocked nose and sinuses and helps maintain healthy sinuses by removing excess mucus.

COMPOSITION:

Medicinal Ingredient: Sodium Chloride (from sterilized sea water) 2.3%.
Non-medicinal Ingredient: Purified water.

PRECAUTIONS:

- ▶ Do not use in case of hypersensitivity to sea water.
- ▶ Do not share the device with anyone else.
- ▶ Do not swallow the solution.
- ▶ If you or your child experience any discomfort, discontinue using this product and consult a health care practitioner.
- ▶ Instruct children less than 12 years of age how to use the product in order to minimize swallowing.
- ▶ Consult a health care practitioner if symptoms persist or worsen.
- ▶ Consult a health care practitioner if there is no improvement after you have tried to clear the nose, if the discharge is thick and yellow or green, if the skin at the base of the nostrils becomes raw or crusted and/or if there are other signs or symptoms of illness such as fever or ear infection.
- ▶ Do not use if nasal passage is completely blocked or if you, or the child, have an ear infection or blocked ears.

ADVERSE REACTIONS:

A stinging feeling or transitory irritation may be experienced at the beginning of use.

Store between 15 °C and 25 °C.

Preservative-free. Environmentally-friendly.

USE:

Clears blocked nose and sinuses and helps maintain healthy sinuses by removing excess mucus.

COMPOSITION:

Medicinal Ingredient: Sodium Chloride (from sterilized sea water) 2.3%.
Non-medicinal Ingredient: Purified water.

PRECAUTIONS:

- ▶ Do not use if your child is hypersensitive to sea water.
- ▶ Do not share the child's device with anyone else.
- ▶ Do not swallow the solution.
- ▶ If your child experiences any discomfort, discontinue using this product and consult a health care practitioner.
- ▶ Instruct children less than 12 years of age how to use the product in order to minimize swallowing.
- ▶ Consult a health care practitioner if symptoms persist or worsen.
- ▶ Consult a health care practitioner if there is no improvement after you have tried to clear the nose, if the discharge is thick and yellow or green, if the skin at the base of the nostrils becomes raw or crusted and/or if there are other signs or symptoms of illness such as fever or ear infection.
- ▶ Do not use if nasal passage is completely blocked or if your child has an ear infection or blocked ears.

ADVERSE REACTIONS:

A stinging feeling or transitory irritation may be experienced at the beginning of use.

Store between 15 °C and 25 °C. Contents under pressure. Do not place in hot water or near radiators, stoves or other sources of heat. Do not puncture or incinerate container or store at temperatures over 50 °C.

Preservative-free. Environmentally-friendly.

USE:

Clears blocked nose and sinuses and helps maintain healthy sinuses by removing excess mucus.

COMPOSITION:

Medicinal Ingredient: Sodium Chloride (from sterilized sea water) 2.3%.
Non-medicinal Ingredient: Purified water.

PRECAUTIONS:

- ▶ Do not use in case of hypersensitivity to sea water.
- ▶ Do not share the device with anyone else.
- ▶ Do not swallow the solution.
- ▶ If you experience any discomfort, discontinue using this product and consult a health care practitioner.
- ▶ Consult a health care practitioner if symptoms persist or worsen.
- ▶ Consult a health care practitioner if there is no improvement after you have tried to clear the nose, if the discharge is thick and yellow or green, if the skin at the base of the nostrils becomes raw or crusted and/or if there are other signs or symptoms of illness such as fever or ear infection.
- ▶ Do not use if nasal passage is completely blocked or if you have an ear infection or blocked ears.

ADVERSE REACTIONS:

A stinging feeling or transitory irritation may be experienced at the beginning of use.

Store between 15 °C and 25 °C. Contents under pressure. Do not place in hot water or near radiators, stoves or other sources of heat. Do not puncture or incinerate container or store at temperatures over 50 °C.